



SCHUPP
PHYSIO · FITNESS · WELLNESS



EN INSTRUCTIONS FOR USE



Tedavi Basit

Therapy / examination couch



140030



BASIC

Therapieliege STATISCH
4027887THERAPIELIEGES7V



[Aktuelle Gebrauchsanweisung]



Imprint



Manufacturer



DE-MF-000010127

SCHUPP GmbH & Co. KG
Glattal Street 78
D-72280 Dornstetten-Aach
Internet: www.schupp.shop

Production

SCHUPP GmbH & Co. KG
Neunecker Street 18
D-72290 Lossburg-Wittendorf

Copyright protection

The document including all illustrations is protected by copyright. Any use outside the limits of copyright law without the consent of the manufacturer is prohibited and liable to prosecution.

This applies in particular to duplications, translations and the storage and processing in electronic systems.

Violations give rise to a claim for damages and may have criminal consequences.

All rights reserved.

© 2025 SCHUPP GmbH & Co KG

Warranty / Liability



These instructions for use are mandatory before read carefully and completely before putting the product into operation!

For correct use of the medical device, observe all instructions and safety notes!

The instructions for use must always be accessible to all users!

You can call up the current status of the instructions for use from us online:



We reserve the right to make technical changes within the scope of further development of the therapy bed without prior notice. All information is non-binding. Misprints excepted.

We accept no liability for damage and malfunctions caused by operating errors and non-compliance with these operating instructions.





TABLE OF CONTENTS

[Direct link activated \[press the CTRL key and click on the chapter name\].](#)

1.0	General	4
2.0	Safety regulations	4
2.1	Safety signs - Symbols	4
2.2	Applied standards	5
2.3	Intended use	5
3.0	Safety instructions	5
3.1	General	5
3.2	Safety instructions for delivery / unpacking	6
3.3	Safety instructions for commissioning	6
4.0	Warranty / Warranty	7
5.0	Disclaimer	7
6.0	Function and operation	8
6.1	Set-up and installation	8
6.2	Adjustment of the headrest upholstery (1 part)	8
6.3	Paper roll holder	8
6.4	Equipment, optional assemblies	9
6.5	Accessories	9
7.0	Spare and wear parts	10
8.0	Service life, maintenance and care	10
8.1	Maintenance	10
8.2	Personal precautions	10
8.3	Care and cleaning	11
8.4	Surface disinfection	11
9.0	Help with faults	12
10.0	Nameplate	12
11.0	Disposal	13
12.0	Technical data	13
13.0	Declaration of conformity	13
14.0	Contact	16





1.0 General

This therapy table for the professional therapy sector was developed in close cooperation with the requirements and specifications of physiotherapists. Special attention was paid to the safety of the couch, the comfort of the couch as well as durability and service life.

The design and manufacture of the therapy couch comply with the current safety principles of the BfArM and take into account the generally recognised state of the art. The couch does not impair the safety or condition of patients or the safety and health of users or third parties.

"Made in Germany" as well as the use of the highest quality materials guarantee you a long and reliable functionality of the therapy couch when used as intended.

All therapy beds from SCHUPP GmbH & Co.KG are developed in the Black Forest / Baden Württemberg and manufactured with regional suppliers. The therapy beds are assembled in our ISO-certified production facility, thus guaranteeing a high and always consistent quality of workmanship.

2.0 Safety regulations

2.1 Safety signs - Symbols

Safety-relevant sections, instructions or notes are highlighted by the following symbols. Please familiarise yourself with the definitions before reading the instructions for use.



DANGER

Indicates an immediate danger for users, patients and third parties, which can result in death or serious injury.



Warning

Indicates a potentially dangerous situation that may result in a low or moderate injury to the user, patient, third parties or damage to the medical device. or on other objects.



Warning

Warning of danger of trapping and crushing when adjusting the therapy couch



Caution

Indicates a possible danger to property, which may result in property damage.



Always follow the instructions for use



Additional notes, application tips, useful information





2.2 Applied standards

The therapy couch has been developed and manufactured according to state-of-the-art technology and in compliance with national/international guidelines. It complies with the following guidelines:

EU MDR (Medical Device Regulation) 2017/745	This product is a class I medical device according to the EU Medical Device Regulation.
DIN EN ISO 10993-1:2021-05	Biological evaluation of medical devices
DIN EN ISO 15223-1:2022-02	Medical devices - Symbols for use in providing information
DIN EN ISO 20417:2022-03	Medical devices – Requirements for the information to be provided
DIN EN 62366-1:2021-08	Medical devices - Application of fitness for use to medical devices
Machinery Directive 2006/42/EC	Medical hydraulic equipment - General requirements for safety

2.3 Intended use

The therapy bed from Schupp GmbH & Co. KG is a class 1 medical device.

EU MDR 2017/745) and is intended exclusively for professional use by medically trained personnel and instructed users for the proper positioning and support of patients during diagnostic or therapeutic measures in physiotherapy, chiropractic, osteopathic, or general medical treatments of patients in the rooms provided for this purpose.

“Static medical couch for diagnostic and/or therapeutic purposes”

The Schupp company does not give any guarantee regarding the suitability of the therapy couch for a specific therapeutic or diagnostic purpose. The therapist determines the appropriate form of application.

The therapy couch is neither intended nor equipped for hydrotherapy nor for domestic or radiological applications.

3.0 Safety instructions

3.1 General

For your own safety as well as the safety of the patient and third parties, familiarise yourself with the limits and dangers before using the unit for the first time.

Observe all safety and warning instructions! Also observe the safety and warning instructions when using equipment together!



These operating instructions contain important information on safe handling, operation, care and maintenance.

The instructions are part of the therapy bed and must always be kept handy for all users.

In the event of a change of ownership or location, these instructions must be handed over with the couch. Failure to comply with these instructions will result in the loss of warranty and guarantee claims!

Observe the additional regional and country-specific rules and regulations for occupational safety and accident prevention that apply to the place of use.



**Caution**

The operator may only use the therapy bed for therapy and treatment within its intended purpose.

The operator may only allow qualified personnel or persons instructed and authorised in work safety and accident prevention regulations to work on the therapy bed.

The operator must ensure that all users have read and understood the instructions, warnings and safety instructions in this manual and are sufficiently informed about the mode of operation.

The user must ensure that children are always supervised and do not stay in the area of the therapy couch.

The therapy couch / pads are not sterile. Open wounds or skin irritations must not come into contact with the padding material.

When using additional devices (e.g. red light or radiant warmer), the user must ensure that the therapy table cannot be moved unintentionally or accidentally. Safety distances must be strictly observed.

3.2 Safety instructions for delivery / unpacking



The therapy couches are 100% checked visually and for mechanical function before dispatch. The couch has left the factory in perfect and operational condition.

Remove the transport securing strap under the head section and in the frame!

Be careful when opening the packaging! Do not use pointed or sharp objects, the cushion could be directly damaged.

Immediately upon receipt of the shipment, check for completeness of the delivery as well as packaging damage. In the event of damage, this must be claimed immediately from the carrier and must be noted immediately on the shipping documents.

If damage to the medical device is detected, this must be reported in writing to Schupp GmbH & Co. KG within 2 working days.

After this period, warranty claims arising from transport damage can no longer be asserted.

3.3 Safety instructions for commissioning

**Warning**

Treatments may only be carried out on a therapy couch that is in perfect safety condition; there is no danger from electrical energy.

The couch was designed exclusively for use under normal conditions indoors.

The therapy couch should be placed on a firm and level surface with adequate free space for the user.

Neither the therapist nor the patient may use the head section as a sitting, kneeling or climbing aid. abuse.

Never exceed the maximum static load capacity of:

150kg (140030)!

**Caution !**

Regular inspection of the technical condition by the operator is a prerequisite.

Regular monitoring or maintenance of wear parts such as operating and adjustment elements, gas springs and upholstery.

To ensure safety, only use original accessories from Schupp. Incorrect accessories can lead to material damage to the bed or to injuries!

Repair and maintenance only by the manufacturer or personnel authorised by the manufacturer.

The lounge may not be modified without the written permission of the manufacturer. Conversions and technical modifications are not permitted.

When used outside the intended purpose or when using additional devices, it is the responsibility of the operator to comply with the limit values for the safety of medical electrical systems according to DIN EN 60601-1 and following.



The customary national laws / regulations must be observed when operating this therapy bed. Observe the necessary safety checks according to MPBetreibV (Medical Devices Operator Ordinance), BetrSichV (Operational Safety Ordinance) and DGUV (German Social Accident Insurance).





4.0 Warranty / Warranty

Schupp GmbH & Co KG guarantees you a therapy couch free of material and processing defects. The couch was thoroughly tested before dispatch and 100% approved in the final production test.

The guarantee period is 60 months for the couch frame, 12 months for upholstered parts, 24 months for all other components of the therapy couch, with the exception of wear parts, and begins at the time of delivery. Please be sure to keep the invoice as proof of purchase.

Parts or components of the couch that prove to be defective within these periods can be sent to the service department for compensation after prior consultation. Product liability ends 10 years after the product has been placed on the market. The manufacturer is no longer obliged to provide replacement parts or accessories.

The General Terms and Conditions for Deliveries and Services as well as the guarantee and warranty provisions of Schupp GmbH & Co. KG [current version at: www.schupp.shop] shall apply.

The therapy bed may only be repaired or modified by the manufacturer or by a person or company authorised by the manufacturer! Otherwise the warranty claim will expire!

5.0 Disclaimer

We expressly draw your attention to the fact that no liability is accepted for personal injury and/or damage to property caused by failure to observe the instructions for use, by incorrect or negligent operation, maintenance, servicing and repair or by improper use of the therapy bed or its accessories, or by exceeding the performance limits of the bed. This also applies to the effects of foreign bodies, modifications and additions to or conversions of the couch which impair or could impair safety.





6.0 Function and operation

6.1 Set-up and installation

Place the therapy couch at the desired treatment location and remove the protective foil from the couch. Do not use a cutter, knife or similar pointed tools for this. The upholstery could be damaged.

The packaging is made of environmentally friendly and recyclable materials and can be disposed of properly at a collection point. Also note when setting up:

- Carry the couch only by the lower metal frame, never use the headboard as a carrying handle!
- Remove the transport securing tape under the head section and in the frame before commissioning!
- Couch must stand firmly on the floor for use!
- Note the width of the couch - Do not push the couch against a door frame! Upholstery can burst open!

The stability in case of small unevenness in the floor can be improved by unscrewing the level adjustment on the feet of the couch frame.



6.2 Adjustment of the headrest upholstery (1 part)

You can raise or lower the headrest upholstery steplessly by pressing the operating lever under the headrest upholstery in the direction of the upholstery surface and simultaneously moving the headrest up and down.

After releasing the actuating lever, the gas pressure spring, which is blocked in the direction of tension and compression, holds the headrest cushion securely in its newly adjusted position.

The nose slot insert can be inserted or removed depending on the application.



Warning.

- Moving parts - Do not press the operating lever of the head section upwards in an uncontrolled manner!
- Neither the therapist nor the patient may sit, kneel or stand on the head section of the couch!
- The head section may be loaded with a maximum of 30kg!

6.3 Paper roll holder

The therapy couch can be equipped with the option "paper roll holder".

The paper roll holder is mounted under the cushion or on the frame of the foot section. This is possible on both footrests. The holder can accommodate paper rolls with a width of up to 59cm.





6.4 Equipment, optional assemblies

Bed type	TEDAVI Basit
Item no. :	140030
Medical device	●
Load capacity 150kg	●
Cushion	2
Cushion width 65 cm	●
Cushion width 80 cm	○
Imitation leather, 6cm, durable composite structure	●
Imitation leather, 8cm, Comfort de Luxe padding, extra soft	
Head section incl. nose slit adjustable with GDF, +50° / -70°.	●
Nose slit insert	●
Lying surface 2 parts	●
Lounger height 65 cm	●
Lounger height 70 cm	○
Lounger height 80 cm	○
Upholstery colours	37
Special upholstery, motif, embroidery	
Paper roll holder	○
Lounger Protective Cover 600x400	○

- Standard equipment
- optional

6.5 Accessories

Bed type	Item no. :	TEDAVI Basit
Item no. :		140030
Gas pressure spring head section	402065	●
Protective cover 600x400	190999	●

Accessories for couch type





7.0 Spare and wear parts

Repairs must not affect the safety of the couch. Therefore, they may only be carried out by the manufacturer or persons authorised for this purpose. Only original spare parts from the manufacturer may be used for repairs.

If you notice any defects or wear on your therapy bed, please inform us using the contact details provided (see last page). This also applies to type plates and labels.

When contacting us, always state the couch type, the article number and the serial number. You will find this data on the type plate on the frame of the foot section end of the couch (see chapter 10).

Our service staff will be happy to help you define the fault and select the right spare and wear parts such as arm, foot, head or body pads, foot plugs and operating parts: Arm, foot, head or body pads, foot plugs and operating parts.

8.0 Service life, maintenance and care

The therapy couch you have purchased is designed to last an extremely long time. There are no limiting factors with a static couch. Depending on the maintenance and care of the table, the maximum service life of 10 years or 100,000 cycles for electrically / hydraulically designed tables can be significantly exceeded.

8.1 Maintenance

In order to ensure the safety of the patient and user and to protect the therapy table from damage, expert use and regular maintenance in accordance with DIN 31051 are required. We recommend that you have the table serviced every 1-2 years by professionally trained Schupp employees or by authorised specialist dealers. As part of the maintenance, all prescribed safety checks are carried out in accordance with the current safety regulations:



Testing of medical technology in accordance with DIN EN 62353 (VDE 0751-1), annually and in case of change

- ▶ BetriebSichV (Ordinance on Industrial Safety and Health)
- ▶ MPBetreibV (Medical Devices Operator Ordinance)
- DGUV (German Social Accident Insurance); Regulation 3 (formerly BGV A3)

carried out.

8.2 Personal precautions

To maintain functionality and comfort, it is recommended to check the therapy couch regularly by visual inspection (at least 4 times per year). You can take some precautions yourself to prolong the life of the couch. These are:

Guides and joints

The therapy table you have purchased is designed for an extremely long service life. The guides and joints are therefore low-maintenance. If necessary, the joints can be lightly lubricated with a fine mechanical oil. Also visually check the screw connections, weld seams, safety pins and bolts for completeness and mechanical integrity.



Caution.

- ▶ Regular inspection/maintenance as well as compliance with technical tests do not relieve the user of his responsibility for the safety of the patient and the functional efficiency of his treatment systems!



- ▶ No tools are to be used during the visual inspection. In case of any abnormalities, inform the service department immediately!





8.3 Care and cleaning

The upholstery of the therapy couch must be treated very gently despite the highest quality of leather. The surfaces should be cleaned or disinfected after each treatment.

In case of light soiling, clean the upholstery immediately with a soft cloth, a household sponge or a soft household brush in combination with lukewarm soapy water. The use of a commercially available microfibre cloth is particularly recommended.

For heavy soiling, soapy water can be used in concentrated form. Repeat brushing if necessary. After cleaning, wipe with clean water until there is no more cleaner on the artificial leather.

Moisture residues must be dried off with a soft cloth!

To prevent stains from penetrating the material, they should be removed immediately. The use of abrasive cleaning agents (scouring agents) or solvents (nitro, acetone, etc.) will damage and after a short time lead to brittleness and discolouration of the artificial leather surface.



Warning.

- ▶ The upholstery must not be cleaned with alcoholic or chemical solvents, e.g. thinner, petrol or similar!
- ▶ Do not use chlorides, synthetic cleaning agents, wax polishes or aerosol sprays for cleaning.



Cleaning and disinfection of the couch, including the upholstery, may only be carried out with a suitable and approved disinfectant cleaner.



Discolouration - The manufacturer is not responsible for material discolouration of the synthetic leather due to contact with denim or other textiles.

8.4 Surface disinfection

The cleaning and disinfection of the couch, including the upholstery, may only be carried out with suitable disinfectant cleaners (observe application concentration!).

The currently released disinfectants / disinfectant cleaners vary depending on the manufacturer and leather type. Our sales and/or service staff can provide you with the latest release documents if required. You can also call up the current status online:



[SKAI – Evida, Palma, Plata, Pandoria, Tundra]



[AKV – Amalfi, Peri, Sileather]





9.0 Help with faults

Symptoms	Possible causes	Measures
Cushions cannot be aligned mechanically	If the couch has not been used for a longer period of time, the supporting gas pressure springs may be stiff. Gas strut defective.	In this case, the lever of the gas pressure spring must be pulled completely upwards. With the other hand, move the lying surface completely up and down several times. Exchange gas strut
Problem cannot be fixed	unknown	Contact the Schupp service department (see last page). Make sure you pay attention to the following features of your therapy couch: Type, part number Serial number You will find this data on the type plate on the frame of the foot section end of the couch.

10.0 Nameplate

Applied symbols:



	Medical device
	Article number
	Type no. - Date of manufacture - Serial number
	Unique Device Identification
	Manufacturer, address
	Attention ! Warnings present
	Read instructions for use carefully, observe warnings
	Product for internal use only
	CE conformity marking (see chapter 13.0)
	Product must not be disposed of with household waste
	Product compliant according to ROHS II Directives 2011/65/EU





11.0 Disposal

Do not dispose of this appliance with normal household waste !

In order to ensure environmentally friendly disposal of the therapy bed, optional assemblies and spare parts at the end of their service life or after decommissioning, please contact the Schupp service department. We will be happy to help you with the disposal of your therapy bed.

If you dispose of the couch yourself, the following information is important for you:

- Couch frame , consists of lacquered steel
- Upholstery parts, consist of a wooden board (MDF), composite foam, PU foam, imitation leather cover.

The resulting waste must be disposed of or recycled in a manner that is safe for people and the environment^(*) . Please always inform yourself about the regulations in force in your country and also about the local regulations for the disposal of old appliances and accessories.

^(*) Watch out for physical hazards such as sharp edges when dismantling / disposing of the unit.

12.0 Technical data

Medical devices class	EU MDR	Class 1 medical device
Dimension lying surface	Length	195 cm
	Length Headboard upholstery	45 cm
	Length Body part padding	149 cm
	Wide	65 , 70 , 80cm
	Reference	Imitation leather
	Base plate	Wood fibre board P2 - 19 mm E1 [EN13986:2004, A1:2015]
	Upholstery: Standard	6 cm cushion thickness Foam composite, padded edges
The 2-piece lying surface includes	Body part padding	1 part
	Headboard upholstery	1 part Nose slit opening and cover Manually adjustable with gas pressure spring
	Adjustment range	+50° to -70°
Lying surface	Height	65, 70, 80 cm
Resilience (load evenly distributed over the entire lying surface)	Static load	Max. 150kg (140030)
Weight	2-piece 65cm	40 kg
	2-piece 80cm	44 kg
Accessibility		YES, recommendation SST, SSU Tripod
Application climate	Temperature	+5 to +40 °C
	Humidity	20 % to 90 % RH (at 30 °C)
Transport climate	Temperature	-10 °C to +50 °C
	Humidity	20 % to 90 % RH (at 30 °C)

13.0 Declaration of conformity



**EU-Konformitätserklärung**

EC-Declaration of Conformity
CE-Déclaration de conformité
CE-Dichiarazione di conformità

THIS DECLARATION OF CONFORMITY IS ISSUED UNDER THE SOLE RESPONSIBILITY OF THE MANUFACTURER

**Name, Hersteller**

Name, manufacturer | nom, producteur | nome, produttore

SRN | Single Registration Number

Adresse

Address | adresse | indirizzo

Wir erklären in alleiniger Verantwortung, dass das Medizinprodukt

We declare under our sole responsibility that the medical device
Nous déclarons sous notre seule responsabilité que le dispositif médical
Dichiariamo sotto la nostra esclusiva responsabilità che il dispositivo medico

Typ, Modell

type, model | tipo, modèle | tipo, modello

Basic-UDI | Unique Device Identification

Zweckbestimmung

intended purpose
usage prévu
destinazione d'uso

die grundlegenden Sicherheits- und Leistungsanforderungen erfüllt.

Meets the essential safety- and performance requirements
Répond aux exigences essentielles de sécurité et de performance
Soddisfa i requisiti essenziali di sicurezza e prestazione

Schupp GmbH und Co. KG

DE-MF-000010127

D - 72280 Dornstetten | Glattalstrasse 78
Germany | Allemagne | Germania

Therapieliege, statisch

Therapy couch, Treatment table, static
Table de physiothérapie, statique
Lettimi per terapia, statico

Tedavi

4027887THERAPIELIEGES7V

Medizinische Liege zur Positionierung des Patienten im Rahmen von Diagnosen und/oder zu Therapiezwecke

Medical couch for positioning patients for diagnostic and/or therapeutic purposes
Table médicale pour le positionnement du patient dans le cadre de diagnostics et/ou à des fins thérapeutiques
Letmino medico per il posizionamento del paziente nell'ambito di diagnosi e/o a fini terapeutici

EU-MDR 2017/745 [Anhang 1]

EU-MDR 2017/745 [annex 1] | [annexe 1] | [appendice 1]

Konformitätsbewertung | Assessment details | Evaluation de la conformité | Valutazione della conformità**Konformitätsbewertungsverfahren**

Conformity assessment procedure
Procédure d'évaluation de la conformité
Procedura di valutazione della conformità

Klasse | class | classe | classe

Klassifizierung nach Anhang VIII

Classification according to annex VIII
Classification selon l'annexe VIII
Classificazione secondo l'allegato VIII

Gültig bis

Valid until | Valable jusqu'à | Valido fino a

Technische Dokumentation

Technical documentation | Documentation technique | Documentazione tecnica

Post Market Surveillance System

Post market surveillance | Surveillance post-marché | Sorveglianza post mercato

wurden erstellt und sind implementiert.
have been created and are implemented.
ont été créés et sont mis en œuvre.
sono stati creati e sono implementati.

EU-MDR 2017/745 [Artikel 52 Absatz 7, Artikel 19]

EU-MDR 2017/745 [article, article, articolo 52, paragraph, paragraphe, paragrafo 7] | [article, article, articolo 19]

Klasse 1 | Class 1 | Classe 1 | Classe 1

Regel 1

Rule 1 | Terme de règle 1 | Regola 1

26.05.2028

EU-MDR 2017/745 [Anhang II und III]**EU-MDR 2017/745 [Artikel 83-85]**

EU-MDR 2017/745 [annex, annexe, appendice II + III]
EU-MDR 2017/745 [article, article, articolo 83-85]

Erstellt: 20251120 | SCHUPP GmbH & Co. KG | D - 72280 Dornstetten | Glattalstrasse 78

Karin Schupp

Geschäftsführerin | Managing Director | Présidente Directrice Générale | Direttore Generale

20251120-07 | SCHUPP GmbH & Co. KG | D - 72280 Dornstetten | Glattalstrasse 78 | T: +49 (0) 7443/243-0 | info@schupp-gmbh.de | www.schupp.shop



**EU-Konformitätserklärung**

EC-Declaration of Conformity
CE-Déclaration de conformité
CE-Dichiarazione di conformità

THIS DECLARATION OF CONFORMITY IS ISSUED UNDER THE SOLE RESPONSIBILITY OF THE MANUFACTURER



Typ, Modell type, model type, modèle tipo, modello	Artikel-Nummer Item number numéro d'article numero articolo	UDI-DI Unique Device Identification - DI
TEDAVI TEDAVIbasit	140030	(01) 04027887000332 





14.0 Contact

Customer Service,
Maintenance,
Spare parts,
Repair or in warranty cases

You can reach our service under:

Phone +49 (0) 7443 / 243 286
Fax +49 (0) 7443 / 243 8286
Email service@schupp-gmbh.de
Internet www.schupp.shop

You can reach our sales department at:

Hotline +49 (0) 800 / 7248 770
(free of charge from the German fixed network)
Phone +49 (0) 7443 / 243 399
Fax +49 (0) 7443 / 243 255
Email vertrieb@schupp-gmbh.de
Internet www.schupp.shop

You can reach our CH-REP under:

CHRN-AR-102686023
Simon Keller AG, Lyssachstrasse 83, 3400 Burgdorf
Telefon +41 (0) 34 4200830
E-Mail m.rohn@simonkeller.ch

Visit us on site:

Schupp GmbH Co KG

Glattal Street 78 | 72280 Dornstetten
Head office | Production of preparations and personal care products

Schupp GmbH Co KG

Neunecker Street 18 | 72290 Lossburg-Wittendorf
Sales | Showroom |
Production of equipment for physical therapy

Schupp GmbH Co KG

Schleifmattstraße 16 | 77716 Haslach / Kinzigtal
Upholstery | Sewing

Visit our webshop:

www.schupp.shop

Web Support:

Phone +49 (0) 7443 / 243 300
Email online@schupp-gmbh.de

